



People.Health.Care.

# Debrisoft® Prepare to heal

To get the most out of Debrisoft® every part of the wound bed needs Debrisofting for **at least 2 minutes** to remove:



## Infection/Biofilm

Use 5 times over 2 weeks in conjunction with an antimicrobial dressing and reassess



## Devitalised tissue and slough

Use until wound bed is clean and devitalised tissue has been removed




## High levels of exudate

Use at every dressing change until exudate levels reduce




## Dry skin and hyperkeratosis


Use when required to remove skin plaques to allow emollients to be absorbed by healthy skin

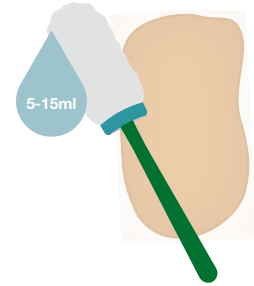

For wounds less than 10cm x 10cm use Debrisoft® Lolly

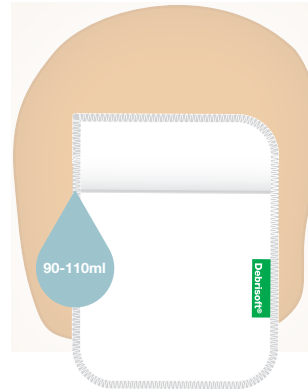

Quicker than ever for faster wound coverage with Debrisoft® Pad (13cm x 20cm) – for larger wounds bigger than 13cm x 20cm

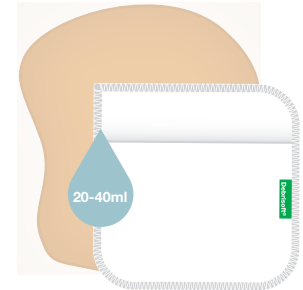

For wounds less than 13cm x 20cm use Debrisoft® Pad (10cm x 10cm)

## Do's

### Moisten Debrisoft® with amount shown in the diagram

- ✓ Wash off any emollients before using Debrisoft®
- ✓ Debrisoft® pad is for single use only
- ✓ Following the use of Debrisoft® pad, reassess the wound/skin before selecting an appropriate dressing/therapy
- ✓ Use Debrisoft® for at least 2-4 minutes
- ✓ Larger areas may take longer to treat, depending on the amount of slough, debris or hyperkeratosis present
- ✓ When addressing biofilm, Debrisoft® should be used on multiple occasions over a short time

## How to use Debrisoft®


- Step 1:** Remove all emollients where applicable
- Step 2:** Moisten Debrisoft® with tap water or saline
- Step 3:** With gentle pressure use circular motions on the wound bed and long sweeping strokes on the skin
- Step 4:** Ensure a new pad or lolly is used for each wound when moving from wound to skin

Debrisoft® is a registered trademark of L&R.

## Dont's

### DO NOT OVERSATURATE

- ✗ Debrisoft® must not be used as a wound dressing
- ✗ Do not cut Debrisoft®
- ✗ Debrisoft® should not be used if there is a known sensitivity to any of the components of the product
- ✗ Do not use as a washing tool

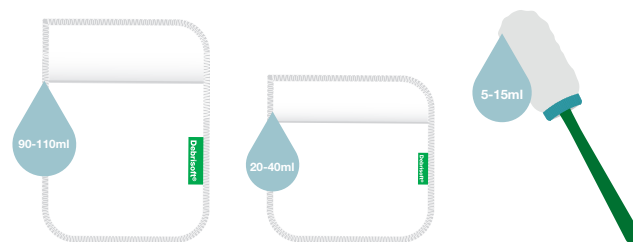

Available on Drug Tariff (FP10/GP10) in the UK



For further guidance please call our Customer Services Team on: **08450 606707** or visit: **[www.Lohmann-Rauscher.co.uk](http://www.Lohmann-Rauscher.co.uk)**



M1932 V1.6