

Suprasorb® P sensitive NEW

Advantages

Efficient exudate management

- Promotes quick absorption of exudate into the dressing
- Maintains moist environment

Dynamic protection

- Water and bacteria-proof
- Permeable to water vapour

Softness & comfort

- Adheres gently
- Limits the risk of adhesion to the wound during dressing changes
- Comfortable thanks to absorbent PU foam



Silicone wound contact layer suitable for **fragile skin**



Efficient exudate management reduces the **risk of maceration** and promotes wound healing



Minimises **pain** to the patient and **trauma** to the wound and skin during dressing removal or repositioning




Can be worn **in the shower**

Easy application:

With **2 to 3** liners provides easy application and avoids touching adhesive with gloves



People.Health.Care.

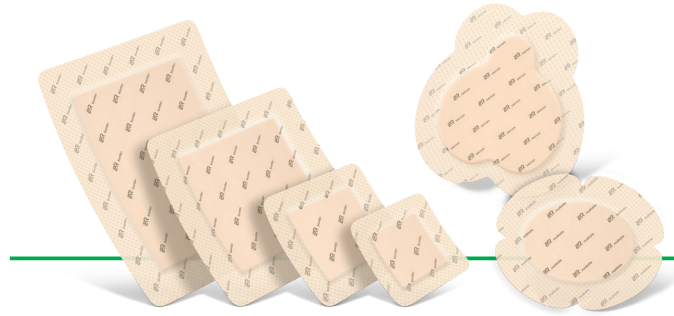

The right shape for every task

The products border and border lite are available in different formats and sizes

Suprasorb® P sensitive NEW

Sterile, individually sealed

Size dressing (cm)	Size wound pad (cm)	REF	Shipping units
border			
7.5 x 8.5	4.5 x 5	139 068	10/200
10 x 10	6.2 x 6.2	139 061	10/200
12.5 x 12.5	8.5 x 8.5	139 062	10/200
15 x 15	10.5 x 10.5	139 069	10/150
20 x 20	15 x 15	139 070	10/100
border lite			
5 x 5	2.5 x 2.5	139 080	10/240
5 x 10	2.5 x 6	139 081	10/200
7.5 x 8.5	4.5 x 5	139 086	10/200
9 x 15	5 x 10.5	139 087	10/200
10 x 10	6.2 x 6.2	139 083	10/200
10 x 20	5 x 15	139 088	10/150



5 037696 714237

Call our Customer Solutions line: **08450 606 707**
or visit our website at: www.Lohmann-Rauscher.co.uk

L&R Medical UK Ltd, 1 Wellington Court, Lancaster Park, Needwood, Burton on Trent, Staffordshire DE13 9PS

M2722 V1.1




People.Health.Care.

Suprasorb® P sensitive NEW Silicone foam dressing



www.Lohmann-Rauscher.com

Versatile protective dressings for low to moderately exuding wounds

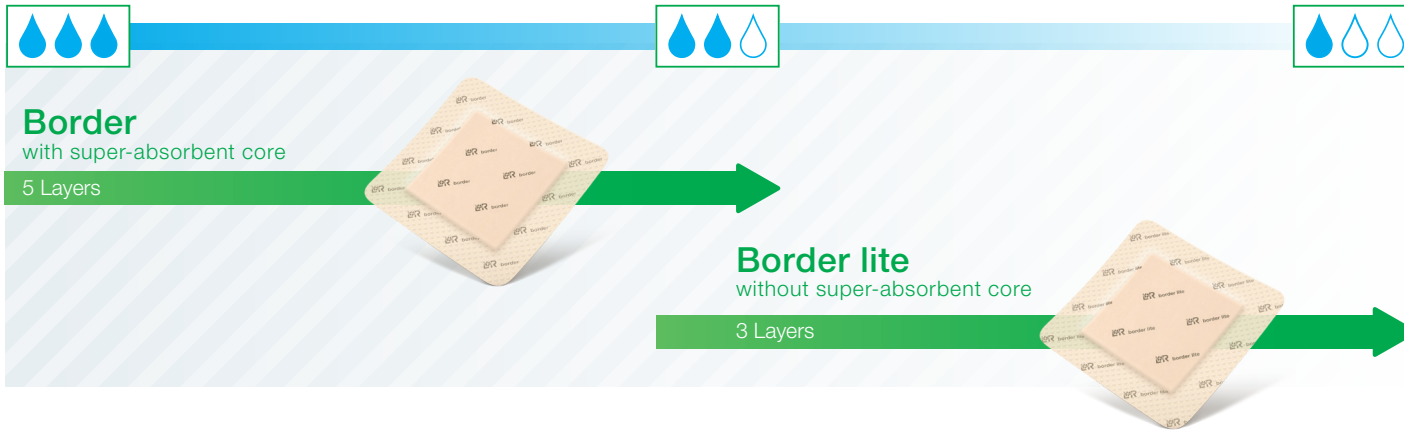

Five layers of softness for sensitive skin

Protective backing:

- Breathable backing with high moisture vapour transmission rate (MVTR)
- Clear backing allows monitoring of exudate without removal
- Smooth and conformable
- Waterproof and can be worn in the shower
- Bacterial barrier

Suprasorb P sensitive border with super-absorbent core:

- Super-absorbent polymer fibres
- Good absorption capacity
- Helps reduce the risk of maceration

Non-woven layer:

- Helps to evenly distribute exudate maximising absorption efficiency

Absorbent PU foam*:

- Efficient exudate management
- Offers comfort and a cushioning effect





OptiSil silicone wound contact layer:

- OptiSil Technology combines optimal perforations with silicone oil
- Enables effective absorption of exudate into the dressing
- Soft and thus minimises pain during dressing removal

* Polyurethane foam

Suprasorb® P sensitive NEW

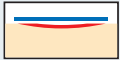Indicated for superficial to deep wounds*

 Superficial wounds	 Deep wounds*
 Acute wounds	 Chronic wounds

Chronic wounds



Leg ulcers



Diabetic foot ulcers




Pressure ulcers

Surgical wounds



Surgical staples/sutures

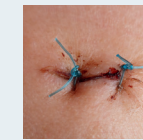

Minimally invasive surgery



Wound dehiscence

Traumatic wounds



Skin abrasions



Skin tears



Skin graft donor and transplant sites

* When used in combination with a primary dressing